

LUEDE EXOPHTHALMOMETER

#53503

Anleitung:

Die Basis des transparenten Messinstruments wird mit der Einkerbung direkt auf den temporalen Orbitarand aufgesetzt. Dabei sollte das Exophthalmometer leicht nach oben gedreht werden, so dass die höchste Ablesung am Hornhautvorderrand erfolgt. Der Patient sollte direkt nach vorne schauen und die Untersuchung in einem gut beleuchteten Umfeld stattfinden. Als Norm gilt der Messwert des Gegenauges. Dies bedeutet, dass beide Augen gleiche Messwerte aufweisen sollten.

Reinigen Sie das Exophthalmometer mit einem weichen Tuch und milder Desinfektionslösung ohne Alkohol.

Instructions:

The base of the transparent rule is placed directly against the external orbital canthus.

The ruler must be tilted slightly upward in order to obtain the highest reading at the apex of the cornea. The patient should face good light and look directly forward. The norms are the other eye, meaning both eyes should have equal findings.

Clean the exophthalmometer with a soft tissue and a mild disinfectant without alcohol.

Vertrieb/Distribution:
TRUSETAL VERBANDSTOFFWERK GMBH
Konrad-Zuse-Str. 15
D - 33758 Schloss Holte-Stukenbrock
Tel: +49 5207 991688-0
Fax: +49 5207 991688-28
Email: info@tshs.eu
Website: www.tshs.eu
Onlineshop: www.eyesfirst.eu

ANL-LUEDE

TRU/A1/12-2014

1/1

MED/CERT

EN ISO13485:2012
+AC:2012